
Vista® CTO

Vista® CT04

VISTA CTO AND VISTA CT04



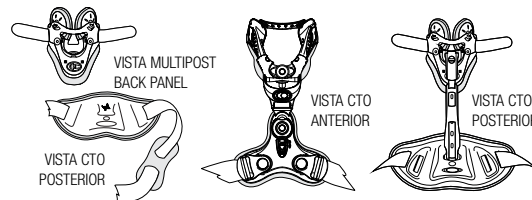
Aspen
MEDICAL PRODUCTS

ASPEN MEDICAL PRODUCTS

6481 OAK CANYON, IRVINE, CA 92618-5202, USA
800-295-2776 · 949-681-0200 · aspenmp.com

VISTA CTO

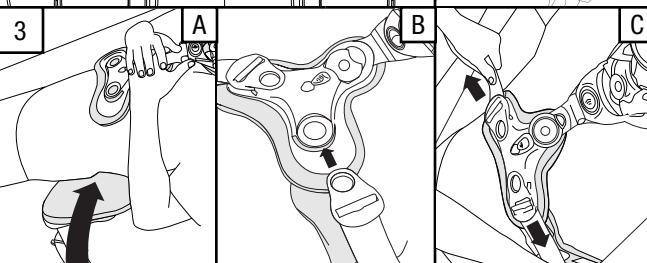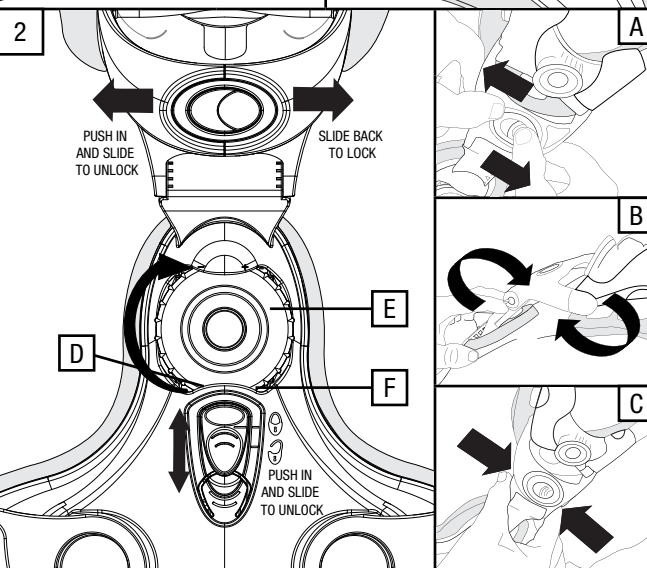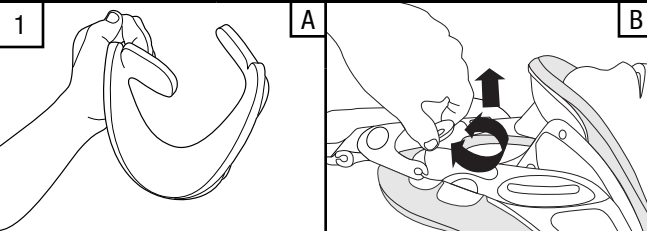ENGLISH

ESPAÑOL

DEUTSCH

ITALIANO

FRANÇAIS

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VISTA ADJUSTMENT

- Pre-form front of the collar.
- Pull and twist the dial to adjust the height on the Vista Collar. (see Vista Instructions at www.aspenmp.com)

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CTO VEST ADJUSTMENT

- Push the button and pull to separate the angle adjustment yoke.
- Determine the desired custom angle.
- Squeeze yoke back together to lock in the custom angle.
- Push down on the secondary lock below the yellow primary lock. Now slide the yellow primary lock down to enable vertical height adjustment.
- Turn the dial until the vest sits at the desired position. When fitting a supine patient, the vest should be positioned as low as possible.
- Slide the yellow button up to engage the primary lock.

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POSITION BACK LUMBAR PANEL

- Slide back panel under lumbar spine.
- Attach the back panel clips to the sides of the vest.
- Pull the straps evenly until the back panel is tightened. Adjust the strap padding for comfort.

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LIMITED WARRANTY
 Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products shall have no obligation under this limited warranty in the event:

(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;

(b) The product is altered;

(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or

(d) The product is not used in accordance with the Aspen Medical Products Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.

This warranty gives you specific legal rights and you may have other rights which vary from state to state. Warning: To ensure proper use, please review all material. A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.