



Caring for Children



Caring for Children



Children need medical products designed specifically for them.

Aspen® understands this and offers five sizes of pediatric cervical collars, allowing you to provide the highest level of patient care.

Five Sizes

The five Aspen® Pediatric Collars automatically conform to provide a custom fit for all children. Collars are also available with a set of replacement pads.

**The Vista® Collar could be used for larger sizes*



PD1

Age 1 to 18 months

Size 21 to 33 in.

Weight 11 to 29 lbs.



PD2

Age 9 to 24 months

Size 29 to 37 in.

Weight 22 to 33 lbs.



PD3

Age 1 to 3 years

Size 33 to 40 in.

Weight 24 to 36 lbs.



PD4

Age 2 to 5 years

Size 35 to 45 in.

Weight 26 to 42 lbs.



PD5

Age 3 to 6 years

Size 37 to 48 in.

Weight 27 to 54 lbs.

Safe for Delicate Skin

Aspen® Pediatric Collars minimize the potential for skin breakdown, while maintaining superior motion restriction. The FlexTabs™ have been designed to spread support across the contact surfaces, eliminating “hot spots” that can lead to skin breakdown.

Gentle by Design

The pads are made of breathable, open cell foam with a soft, hypoallergenic cotton lining which keeps the skin clean, cool and comfortable.

Custom Fit

The PD 4 and PD 5 have an occipital support strap on the back panel that when adjusted provides a custom cradle for the back of the head.

Effective Motion Restriction

The Aspen® Pediatric line is proud to offer 3 pediatric sizes and 2 infant sizes, as only a properly fit collar can provide effective motion restriction.

Confirm a Proper Fit



To aid in application and fitting, these three prompts are printed onto the Velcro strap.

- 1 Chin-Dots hidden?**
The chin piece plastic should be flush with the end of the chin, hiding the “Chin-Dots” indicator on the chin pad.
- 2 Hook and loop straps past the mark?**
Remove all slack. Evenly tighten and re-tighten the straps, insuring the hook and loop straps extend past the green mark for a secure fit and front/back panel overlap.
- 3 FlexTabs™ bent?**
If the collar has been properly sized and tightened, the FlexTabs™ will be “loaded”, or bent, which disperses pressure. Note: The inner trach bar should never touch the throat.

PRODUCT INFORMATION

PEDIATRIC COLLAR

PD1	PD2	PD3	PD4	PD5
983190	983195	983106	983102	983104

PEDIATRIC COLLAR SET One Pediatric Collar and Extra Replacement Pads

PD1	PD2	PD3	PD4	PD5
983192	983197	983126	983122	983124

Back Panels

PD1,2	PD3,4,5
983121	983116
Circumference	Circumference
8-11" 20-28cm	8.5-13" 22-33cm
Replacement Pad	Replacement Pad
983199	983155



Aspen Medical Products
6481 Oak Canyon, Irvine, CA 92618-5202, USA
800-295-2776 • 949-681-0200
aspenmp.com