
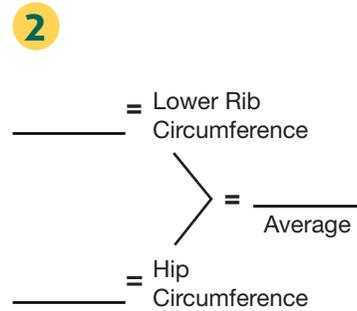


Doctor:	Fitter:
Patient Name:	Date:
Patient #:	Additional Follow-Up Dates:

### THREE STEPS TO SIZING LSO/TLSO

- Determine LSO/TLSO Size (Short or Tall)**  
Measure the distance from the sternal notch to navel.
- Determine Side Panel Circumference**  
Average the lower rib and hip circumferences.
- Determine TLSO Upgrade Kit Size**  
Use the measurement from Step 1

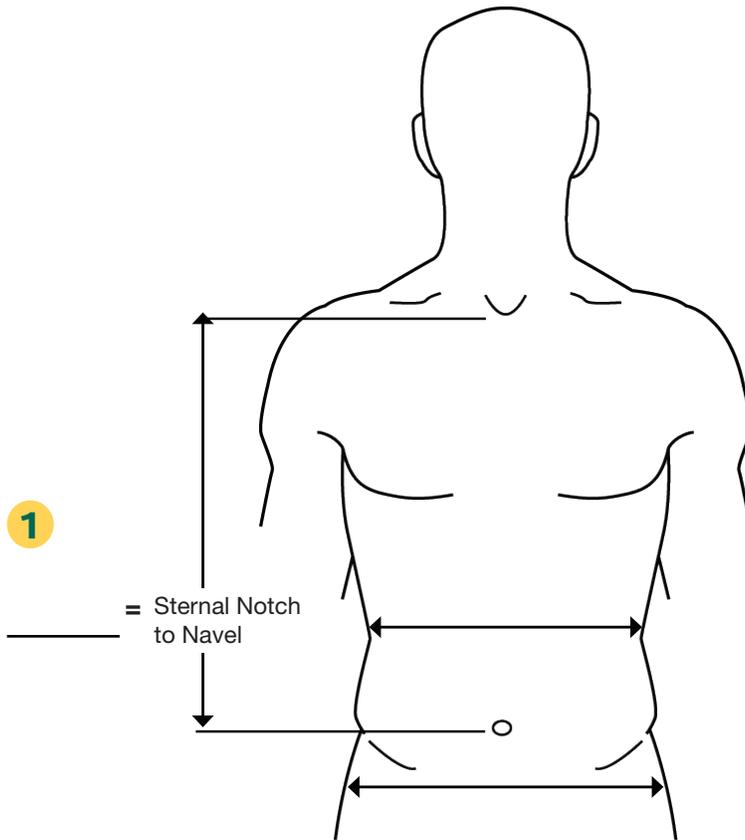

STEP 1 - BACK PANEL	STEP 2 - SIDE PANELS	STEP 3 - UPGRADE KIT
<input type="checkbox"/> <14.5 in (<37 cm) = <b>Short</b> 13 in (33 cm)	<input type="checkbox"/> 26-39 in (66-99 cm) = <b>Small</b>	<input type="checkbox"/> 11-13.5 in (28-34 cm) = <b>X-Short</b>
<input type="checkbox"/> ≥14.5 in (≥37 cm) = <b>Tall</b> 16 in (41 cm)	<input type="checkbox"/> 36-49 in (91-124 cm) = <b>Large</b>	<input type="checkbox"/> 12.5-16 in (32-40 cm) = <b>Short</b>
	<input type="checkbox"/> 32-47 in (81-119 cm) = <b>Small</b>	<input type="checkbox"/> 14.5-18 in (37-46 cm) = <b>Tall</b>
	<input type="checkbox"/> 44-59 in (112-150 cm) = <b>Large</b>	

Aspen TLSO	CAT NO
<input type="checkbox"/> Short / Small	991030
<input type="checkbox"/> Short / Large	991031
<input type="checkbox"/> Tall / Small	991070
<input type="checkbox"/> Tall / Large	991071

Aspen LSO	CAT NO
<input type="checkbox"/> Short / Small	990040
<input type="checkbox"/> Short / Large	990041
<input type="checkbox"/> Tall / Small	990060
<input type="checkbox"/> Tall / Large	990061

TLSO UPGRADE KIT	CAT NO
<input type="checkbox"/> X-Short	991053
<input type="checkbox"/> Short	991051
<input type="checkbox"/> Tall	991050